

CRYO
PROFESSIONAL

Portable cryosurgical system
for professional use only



Convenient, safe, and effective treatment for warts and common skin lesions.

- Easy to use, just takes minutes
- Results similar to liquid nitrogen*
- Low cost per treatment, profitable reimbursements
- Ready-to-use when you need it
- Each kit provides approximately 50+ treatments



TREAT A VARIETY OF SKIN LESIONS IN MINUTES

- Verruca
- Condyloma
- Lentigo
- Keratoses
- Molluscum Contagiosum

See chart on back for indicated lesions




CRYO
PROFESSIONAL




UTM0400 Mixed Kit
UTM0300 2 mm Kit
UTM0200 5 mm Kit

EASY TO USE

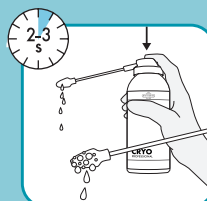


SATURATE

Saturate the applicator by spraying until droplets form

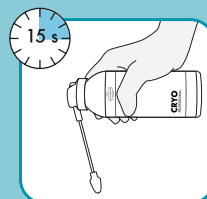

ACCLIMATE

Acclimate the applicator to its effective temperature

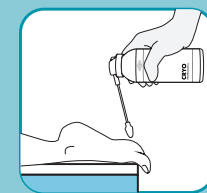

FREEZE

Apply treatment to the lesion as directed

Read full instructions for use thoroughly before handling

DISTRIBUTED BY
Gebauer Company

Visit gebauer.com/cryosurgery for details.

CRYO PROFESSIONAL

Portable cryosurgical system
for professional use only



The convenient, effective, and economical way to treat common skin lesions in minutes.

Cryo Professional provides safe and effective treatment for the reimbursable procedures below. A kit can typically pay for itself within 3-4 procedures. Cryo Professional is ready to use when you need it. Cryo Professional is indicated for the removal of the following lesions:

Indication	Recommended Freeze Time	CPT® Code	2024-B National Payment Amount*	Est. Kit Reimbursement**
Verruca vulgaris (common wart)	40 seconds	17110	\$114.18	\$5,709.00
Verruca plana (flat wart)	20 seconds	17110	\$114.18	\$5,709.00
Verruca plantaris (plantar wart)	40 seconds	17110	\$114.18	\$5,709.00
Actinic keratoses - facial (sun spots)	15 seconds	17000	\$67.91	\$3,395.50
Actinic keratoses - non-facial (sun spots)	40 seconds	17000	\$67.91	\$3,395.50
Seborrheic keratoses	40 seconds	17110	\$114.18	\$5,709.00
Achrochordon (skin tags)	40 seconds	11200	\$92.87	\$4,643.50
Molluscum contagiosum	20 seconds	17110	\$114.18	\$5,709.00
Lentigo - facial (age spots)	15 seconds	17110	\$114.18	\$5,709.00
Lentigo - non-facial (age spots)	40 seconds	17110	\$114.18	\$5,709.00
Condyloma acuminata (genital warts - male)	40 seconds	54056	\$145.80	\$7,290.00
Condyloma acuminata (genital warts - female)	40 seconds	56501	\$193.73	\$9,686.50

* Reference: www.cms.gov/medicare/physician-fee-schedule 2024 Medicare Non-Facility Price. The above information is provided for general reference only and is not intended as reimbursement or legal advice, nor as advice on how to code or submit any specific claim for payment. It is the provider's responsibility to determine and submit appropriate codes according to current requirements and policies of individual payors. Actual payments will vary based on facility type, location, and other factors. Information is subject to change without notice. CPT® is a registered trademark of the American Medical Association.

** Estimate based on 50 single applications per indicated lesion.

Ordering Information

Item Number	Description	Cryo Professional System Contents
UTM0400	Mixed Kit	(2) 80 mL aerosol canisters, (30) 2 mm applicators + (30) 5 mm applicators
UTM0300	2 mm Kit	(2) 80 mL aerosol canisters and (60) 2 mm applicators
UTM0200	5 mm Kit	(2) 80 mL aerosol canisters and (50) 5 mm applicators

Each kit contains: cryogenic aerosol canisters, single-use applicators, and instructions for use

Contains: Dimethyl ether/Propane/Isobutane/Butane



IMPORTANT RISK AND SAFETY INFORMATION:

Cryo Professional is **FLAMMABLE**. Keep away from open flame and other potential ignition sources including diathermy or other medical equipment. Use in a well-ventilated area. Consult a pediatrician before use on patients 4 years old and younger. Do not use if uncertain of diagnosis or if there is a possibility of cancer, or on patients with cryoglobulinemia, circulatory issues, or using immunosuppressant medication. Do not spray cryogen directly on skin or when applicator tip is in contact with skin. Avoid eye contact. Use only as a system with the supplied applicators. Freezing may alter skin pigmentation. Avoid freezing of peripheral arteries. Carefully read instructions for use prior to using this product. **CAUTION:** Federal law restricts this device to sale by and on the order of a licensed healthcare practitioner.



Visit gebauer.com/cryosurgery for details.