

# Give patients the comfort they deserve during minor procedures and injections.

Gebauer's Pain Ease® topical anesthetic skin refrigerant (vapocoolant) can be used in a number of procedures to help improve patient comfort. Pain Ease is FDA-cleared to temporarily control the pain associated with needle procedures and minor surgical procedures, and it begins to work in just 4-10 seconds!

## CONTROL PAIN FOR A VARIETY OF PROCEDURES, INCLUDING:

### Needle Procedures Such As:

- IV starts
- Immunizations
- Blood draws

### Minor Surgical Procedures:

- Suturing or removing sutures
- Lancing boils
- Skin tag removal
- Incision and drainage of small abscesses
- Foreign body removal

Gebauer's  
**PainEase®**  
The INSTANT topical anesthetic skin refrigerant



Pain Ease 3.9oz Medium Stream: 0386-0008-03

Pain Ease 1oz Medium Stream: 0386-0008-04

Pain Ease 3.9oz Mist Spray: 0386-0008-02

Pain Ease 1oz Mist Spray: 0386-0008-01

## 3 STEP USAGE FOR PAIN EASE

### PREP



- Have all the necessary equipment ready
- Prepare the skin per your protocol
- Prepare the patient for the cold effect

### SPRAY



- Hold the can 3 to 7 inches from treatment site, about a can's length away
- Spray steadily 4 to 10 seconds or until the skin begins turning white, whichever comes first
- Do not spray longer than 10 seconds

### PERFORM



- Perform the procedure
- The anesthetic effect lasts about one minute
- Reapply if necessary

*Refer to application instructions for full details*

## IMPORTANT RISK AND SAFETY INFORMATION:

Consult your pediatrician when using on children 4 years old and younger. Do not use on large areas of damaged skin, puncture wounds, animal bites or serious wounds. Do not spray in eyes. Over spraying may cause frostbite. Freezing may alter skin pigmentation. Use caution when using product on persons with poor circulation. Apply only to intact oral mucous membranes. Do not use on genital mucous membranes. The thawing process may be painful and freezing may lower resistance to infection and delay healing. If skin irritation develops, discontinue use. CAUTION: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.



4444 East 153rd Street • Cleveland, OH 44128 • 1-800-321-9348 • [www.Gebauer.com](http://www.Gebauer.com) • 1985.1